
MEBER[®]

IN CASE OF EMERGENCY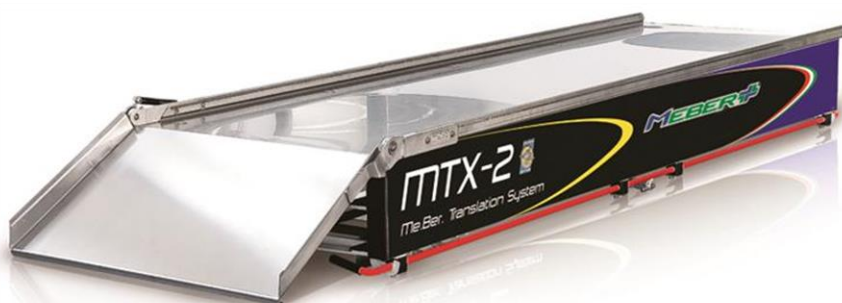

CE

MX · MTX · MX-2 · MTX-2

IT 3 - 12	<i>Manuale delle istruzioni per l'uso e la manutenzione</i>
EN 13 - 22	<i>Use and maintenance instructions manual</i>
DE 23 - 32	<i>Gebrauchs und wartungsanweisung</i>
FR 33 - 42	<i>Manuel d'emploi et de maintenance</i>
ES 43 - 52	<i>Manual de Instrucciones para Uso y Mantenimiento</i>

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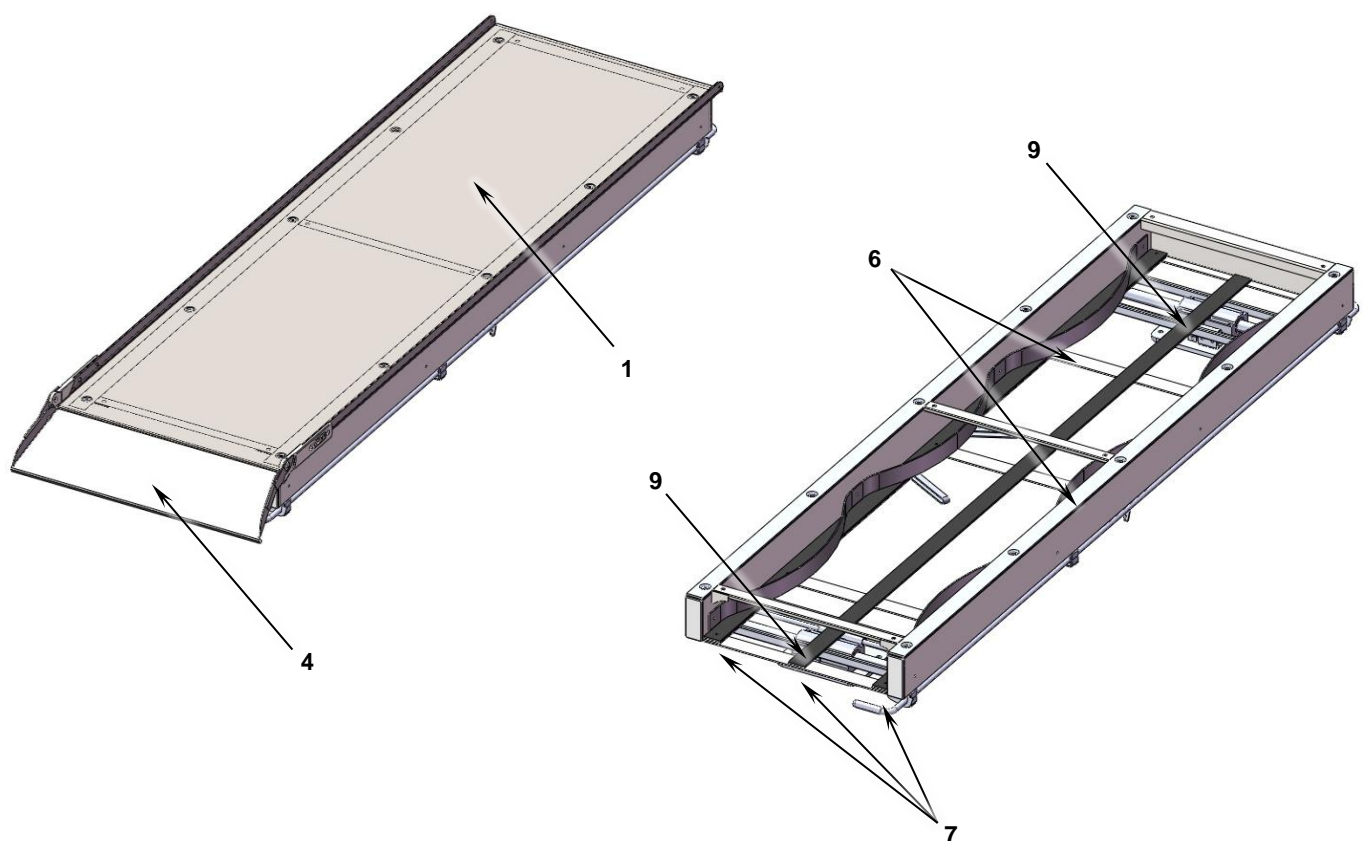
Figura 8

INDEX

1. GENERAL INFORMATION	14
1.1. SYMBOLGY IN USE	14
1.2. WARNINGS	14
1.3. ORGANIZATION AND CONSULTATION OF THE MANUAL	14
1.4. WARRANTY	14
1.4.1. WARRANTEED PRODUCT	14
1.4.2. WARRANTY TERM	15
1.4.3. METHOD OF APPLICATION	15
1.4.4. RETURN OF MATERIAL	15
1.4.5. EXCLUSIONS	15
2. PRODUCT FEATURES AND TECHNICAL DATA	15
2.1. PRODUCT IDENTIFICATION	15
2.2. TECHNICAL DATA SHEET	16
2.3. AVAILABLE MODELS	16
3. SAFETY	17
3.1. APPLICABLE DIRECTIVES AND REGULATIONS	17
3.1.1. CERTIFICATIONS	17
4. INSTALLATION AND INSTRUCTIONS FOR USE	17
4.1. POSITION OF STRETCHER INSIDE THE EMERGENCY VEHICLE	17
4.2. FIXING REINFORCEMENT PLATES	17
4.3. DRILLING THE SUPPORT PLATFORM	17
4.4. FIXING THE SUPPORT	17
4.5. INSTALLING THE FIXING SYSTEMS	18
4.6. PREPARATION	18
4.7. USE	19
5. MAINTENANCE	20
5.1. PERIODICAL INSPECTIONS AND ORDINARY MAINTENANCE	20
5.2. CLEANING	20
5.3. MAINTENANCE REGISTER	20
6. REMOVAL FROM SERVICE AND DISPOSAL	21
6.1. REMOVAL FROM SERVICE	21
6.2. SEPARATION OF MATERIALS FOR DISPOSAL	21
6.3. DISPOSAL	21
7. TECHNICAL SERVICE	21
7.1. ADDRESSES FOR TECHNICAL SERVICE	21
7.2. SPARE PARTS	21

1. GENERAL INFORMATION

1.1. SYMBOLOGY IN USE

	<p>This symbol is used to draw the operator's attention to important warnings regarding patient safety.</p>
	<p>This symbol is used to draw the operator's attention to important general information.</p>

1.2. WARNINGS

This manual is the property of ME.BER. The total or partial reproduction and transmission thereof to third parties by any mechanical, electronic or other type of means, without written authorization on behalf of the manufacturer is forbidden.

This manual is supplied in only one original copy, unless otherwise specified in the order. The purchaser is responsible for providing all concerned persons with a copy thereof.

This manual is supplied together with the support, of which it is an integrant part, and must follow the support even in case the latter is transferred. It should be stored in a safe place and preserved until the support is demolished. In case of loss, a duplicate must be requested from ME.BER., and will be provided, charging the relative expenses.

ME.BER. declines all responsibility for any injury to persons and/or damage to things resulting from failure to observe the instructions contained in this manual.

ME.BER. reserves the right to make all technical and/or commercial modifications deemed useful, at any time and without notice. Therefore, the data and information herein contained may be modified and/or updated.

1.3. ORGANIZATION AND CONSULTATION OF THE MANUAL

This manual has been set out so as to permit the operator to find the information required quickly and easily. The topics covered are all those considered necessary and sufficient for safe use of the support, particularly those that are mandatory and recommended by European Directives, especially as regards safety.

We therefore advise all operators authorized for use to read every section of the manual carefully and to strictly follow all the instructions, contacting the manufacturer for information in case of doubt.

The manual should also be used as a reference document whenever it becomes necessary to review a procedure or operation.

The manufacturer thanks you in advance for any comments that you may wish to provide for the purpose of further improving the format and/or comprehension of the manual.

For publishing reasons, the figures and drawings may differ slightly from their real appearance, without however creating confusion.

Special symbols and **bold** type draw the reader's attention to important information, particularly regarding safety.

The revision index is shown in the bottom right-hand corner of every page.

1.4. WARRANTY

The terms and conditions of the warranty are set forth in the paragraphs that follow, unless otherwise specified in the order.

1.4.1. WARRANTEED PRODUCT

The support has been designed and manufactured to provide many years of use without particular problems; however, in case of defects during the specified period of warranty, ME.BER. will undertake to repair or replace any parts that are broken or present premature wear due to flaws in the materials used, manufacturing defects or imperfect assembly. The warranty does not cover parts that break or present premature wear conditions due to:

- Failure to follow the instructions contained in this use and maintenance manual,
- Failure to perform maintenance or incorrect maintenance,
- Use of tools that are not suitable for ordinary and extraordinary maintenance,
- Tampering with or modifying the support without specific authorization on behalf of ME.BER.,
- Use of non-original spare parts.

For materials bought from external suppliers, ME.BER. grants the purchaser the same warranty as that granted to ME.BER. by the supplier in question.

1.4.2. WARRANTY TERM

The warranty period is 12 months and begins 30 days after the delivery date; this is the only possible term and it is not subject to extension following replacements or repairs carried out during the period of warranty.

1.4.3. METHOD OF APPLICATION

In order to establish the causes of the malfunctions and therefore apply the warranty, the defective components must be sent to ME.BER. At ME.BER's discretion, repairs and replacements under warranty shall be carried out in its workshop or in authorized customer service centers, by third parties or on site. For servicing work carried out on site, the customer shall provide all energy supplies, special tools and auxiliary staff required.

1.4.4. RETURN OF MATERIAL

Before returning parts for which replacement or repair under warranty is requested, it is necessary to have obtained written authorization from the ME.BER. Technical Assistance Department.

Defective components must be properly packaged to avoid damage during transportation and returned ex works to our offices, accompanied by:

- Information from the data plate;
- Code number and position of the component, as specified in the spare parts list (§ 7.2);
- Detailed description of the malfunction and the way it occurred.

Those components for which the warranty is valid will be shipped ex works; replaced components shall remain the property of ME.BER.

1.4.5. EXCLUSIONS

The warranty does not cover materials and components subject to normal wear and those whose service life may not be determined a priori.

ATTENTION: removing the identification sticker from the support automatically renders the warranty null and void.

2. PRODUCT FEATURES AND TECHNICAL DATA

2.1. PRODUCT IDENTIFICATION

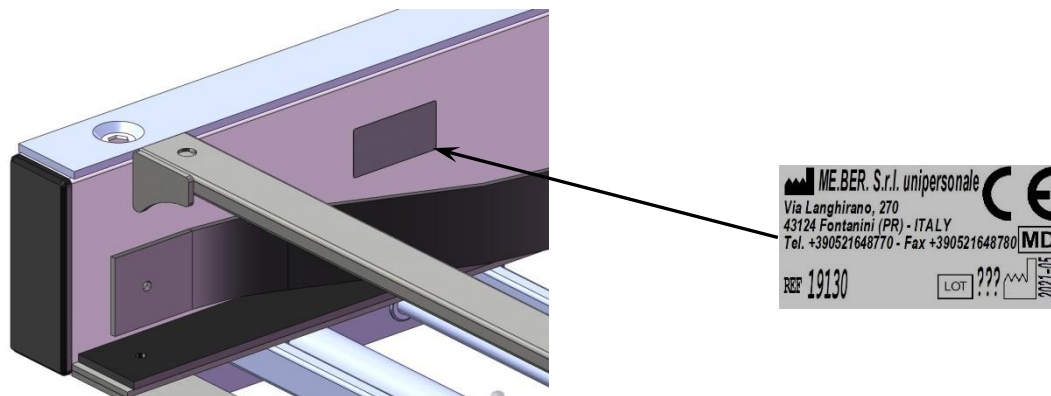
Figure 1

The supports are marked with an adhesive label applied on the internal side of the product: the label contains permanently printed CE mark data (production lot and CE mark).

The adhesive sticker must not be removed. If it is damaged, please request a duplicate. ME.BER. will not recognize a support without a sticker and this will also render the warranty void.

2.2. TECHNICAL DATA SHEET

		19130 MTX	19136 MX	19140 MTX-2	19146 MX-2	19152 MTX-AR-130
A = Total length (with ramp down)	mm	2145	2085	2290	2230	2080
B = Width	mm	605	605	630	630	605
C = Height	mm	195	135	275	215	135
D = Length of loading platform	mm	1922	1922	1922	1922	1922
E = Sideways movement	mm	300	-	300	-	300
Weight	kg	58	41,5	76	59,7	42
MAX capacity	kg	300	300	300	300	300

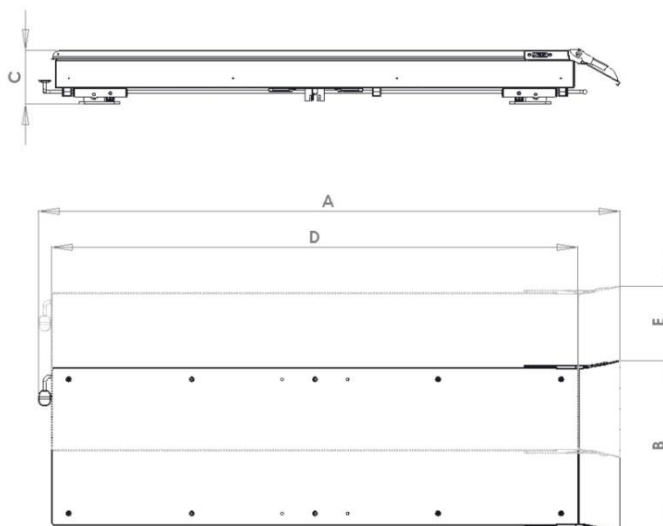
Figure 2

These data are not binding for the manufacturer and may therefore be modified without notice.

2.3. AVAILABLE MODELS

Code	Name	
19130	MTX	Single compartment sideways sliding stretcher support with platform and 180mm ramp
19130-955	MTX-955	Single compartment sideways sliding stretcher support with platform and 180mm ramp and fixing system art. 955
19130-956	MTX-956	Single compartment sideways sliding stretcher support with platform and 180mm ramp and fixing system art. 956
19136	MX	Fixed single compartment stretcher support with platform and 180mm ramp
19136-955	MX-955	Fixed single compartment stretcher support with platform and 180mm ramp and fixing system art. 955
19136-956	MX-956	Fixed single compartment stretcher support with platform and 180mm ramp and fixing system art. 956
19140	MTX-2	Double compartment sideways sliding stretcher support with platform and 350mm ramp
19140-955	MTX-2-955	Double compartment sideways sliding stretcher support with platform and 350mm ramp and fixing system art. 955
19140-956	MTX-2-956	Double compartment sideways sliding stretcher support with platform and 350mm ramp and fixing system art. 956
19146	MX-2	Fixed double compartment stretcher support with platform and 350mm ramp
19146-955	MX-2-955	Fixed double compartment stretcher support with platform and 350mm ramp and fixing system art.955
19146-956	MX-2-956	Fixed double compartment stretcher support with platform and 350mm ramp and fixing system art.956
19152-130	MTX-AR-130	Stretcher support with lateral movement with reduced height without inside compartment and 130 mm ramp

3. SAFETY

3.1. APPLICABLE DIRECTIVES AND REGULATIONS

The supports have been designed and constructed in accordance with the provisions of Regulation 2017/745 on medical devices and are considered **class I** equipment.

3.1.1. CERTIFICATIONS

Conformity with the safety requirements of Regulation 2017/745 was obtained by applying the harmonized norms UNI EN 1789.

Conformity to such regulations has been certified by notified body, as foreseen by normative in force. Certificates are available on our site www.meber.it.

4. INSTALLATION AND INSTRUCTIONS FOR USE

For correct installation, prepare adequate reinforcements at the anchoring points to provide the necessary resistance and avoid damaging the plastic coating. With reference to figures 2 and 3, follow the procedure described below.

4.1. POSITION OF STRETCHER INSIDE THE EMERGENCY VEHICLE

The correct position of the automatic loading stretcher inside the emergency vehicle is one that does not obstruct the work of medical and paramedical personnel.

As far as possible the platform of the emergency vehicle should not have beams or important vehicle parts directly under its platform to prevent damaging them during the drilling phase.

4.2. FIXING REINFORCEMENT PLATES

- Once the position of the support has been determined, before fixing the reinforcement plates carefully check the bottom side of the platform of the emergency vehicle in that specific area;
- Cut two steel plates (10mm thickness) measuring approximately 300 x 200 mm;
- Weld the reinforcement plates to the bottom (external) side of the platform to match the position of the anchoring plates of the support.

4.3. DRILLING THE SUPPORT PLATFORM

- Rest the support onto the ambulance platform in the position previously identified and mark it carefully, so that the hole drilled in the following drilling phase will go through the flat surface of the platform;
- Use a felt pen to mark the position of the eight holes;
- Remove the support and drill holes with Ø10.5mm bit.

4.4. FIXING THE SUPPORT

- Place the support back onto the platform;
- Insert the eight M10 bolts with 8.8 resistance class;
- Tighten the M10 class 8 nuts on the lower part;
- Weld the nuts onto the reinforcement plates;
- Tighten securely.

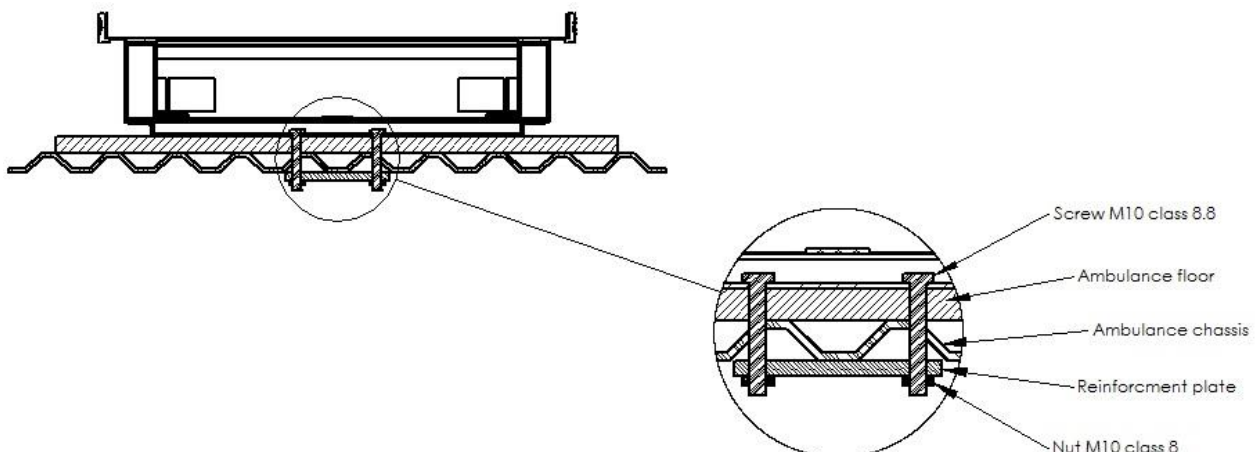
Figure 3 - MX / MX-2

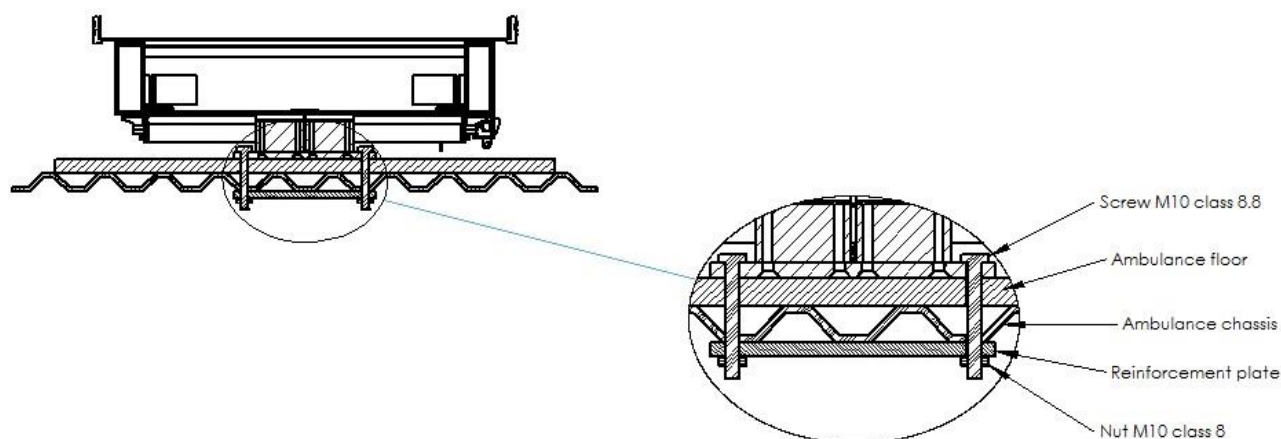
Figure 4 - MTX / MTX-2

4.5. INSTALLING THE FIXING SYSTEMS

The fixing systems must be installed exclusively onto the upper stainless steel platform, without intervening on the support frame or other product parts.

For the installation of fixing systems ME.BER. articles 955-956, follow the instructions provided in the relevant information notes.

The elements under the stainless steel platform must never protrude by more than 20mm.

4.6. PREPARATION

Before using the support, you must be absolutely certain of the efficiency of all its components and devices. To this end, specialized ME.BER technicians perform strict inspections during construction and, for each product, an accurate final test is carried out. However before using the article, it is important to perform the additional preventive checks listed in the following table.

Description of test	Result	
	Positive	Negative
Secure tightening of 8 retaining screws to vehicle	<input type="checkbox"/>	<input type="checkbox"/>
Movement of ramp	<input type="checkbox"/>	<input type="checkbox"/>
Sideways movement (only for MTX / MTX-2)	<input type="checkbox"/>	<input type="checkbox"/>

If the outcome of the test is positive, the device can be used; otherwise you should contact our Technical Support Service immediately.

The above test must be carried out periodically to ensure perfect efficiency of the device and maximum safety conditions.

4.7. USE

SINGLE COMPARTMENT

The compartment can house both a spine board and scoop stretcher

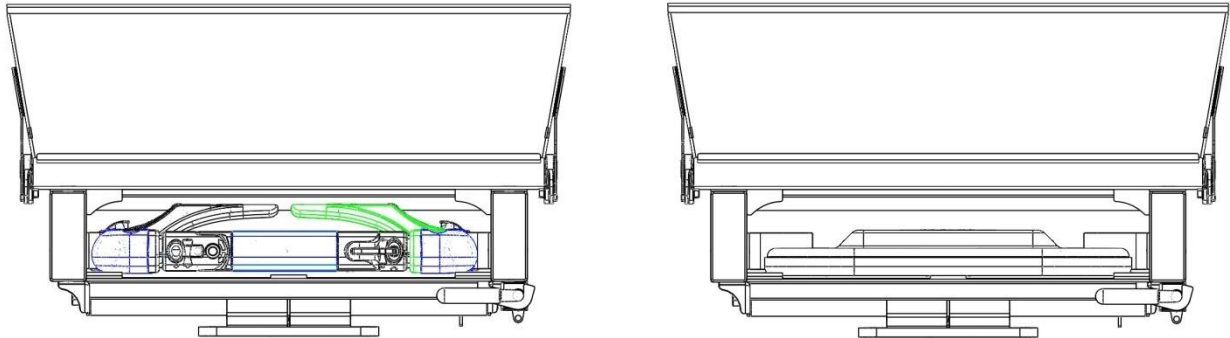
Figure 5

DOUBLE COMPARTMENT

The upper compartment houses the scoop stretcher and the lower compartment the spine board.

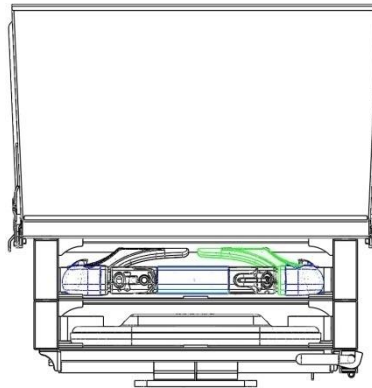
Figure 6

SIDEWAYS SLIDING MODEL

The support can be moved up to 300mm sideways and can be locked in any intermediate position.

Figure 7

To do this, press the front handle down or step on the rear pedal and move the support.

5. MAINTENANCE

5.1. PERIODICAL INSPECTIONS AND ORDINARY MAINTENANCE

In order to preserve the perfect efficiency of the product and ensure, in time, its compliance with safety requirements, it is essential to run periodical inspections and perform the ordinary maintenance.

Periodical inspections must be carried out by the final user. We recommend recording the periodical inspection results on the dedicated sheet included in this manual (§5.3.).

The following chart identifies the **periodical inspections that must be carried out by the final user** and their intervals, which may vary according to the frequency of use of the device.

Type of operation	Frequency (operations/month)		
	≤ 50	51 ÷ 200	> 200
General inspection and functional check of the various devices (§ 4.6.)	6 months	3 months	1 month
Cleaning (see following point)	CLEANING and DISINFECTION must be carried out after EVERY USE		
Check on the tightness of all the screws	4 months	2 months	1 month

Whenever any irregularity in an original product is identified, the product should immediately be taken out of service, until the damaged component is repaired or replaced

To keep the product in perfect working conditions and ensure, in time, its compliance with safety requirements, carry out the ordinary maintenance at least once a year, calling the ME.BER. Technical Assistance Service (§7.1.).

The inappropriate use and maintenance may cause damage to persons and/or things. Carry out the periodical inspections and the ordinary maintenance, according to the instructions of this manual.

For assistance and/or spare parts, call only ME.BER. (§7.1.).

5.2. CLEANING

CLEANING and DISINFECTION must be carried out AFTER USE

Wash and disinfect using a sponge soaked in water and disinfectant. Rinse and dry well.

5.3. MAINTENANCE REGISTER

Date	Description of works	Mechanic

6. REMOVAL FROM SERVICE AND DISPOSAL

6.1. REMOVAL FROM SERVICE

At the end of the useful life, the product must be removed from service. Therefore, proceed to separating the various components.

The identification plate of the product must be destroyed. It is strictly prohibited to use this plate on other products, even ones of the same type.

6.2. SEPARATION OF MATERIALS FOR DISPOSAL

The materials listed below must be separated and stored for recycling or disposal in accordance with environmental regulations.

Material	Components
Stainless steel	Stretcher support platform, ramp
Standard steel	Actuators, pins, frames, supports, bolts
Plastic and rubber	Pads and guards

6.3. DISPOSAL

All these materials should be disposed of according to the current national regulations at the moment of demolition. For correct disposal, use specialised services authorised to collect and treat these materials, or contact the competent authorities for the places and methods.

In any case, do NOT disperse the materials in the environment.

7. TECHNICAL SERVICE

7.1. ADDRESSES FOR TECHNICAL SERVICE

For service under guarantee and/or to request maintenance or repair operations, or for any information, the customer can contact the ME.BER. Service Dept. at:

ME.BER. S.r.l. Unipersonale - Via Langhirano, 270 - 43124 Fontanini (PR) ITALY

Telephone	Fax	e-mail/website
+39 0521 648770 +39 0521 648881	+39 0521 648780	contact@meber.it www.meber.it

Before contacting our Service Department, please consult this manual carefully.

In the event that a service operation is required, the type and description of the problem must be clearly specified so that we can provide the most suitable material.

7.2. SPARE PARTS

When ordering spare parts, please provide: *spare part item number / code / spare part name* from the table on the following page and the quantity required. Example of order form:

Pos.	Code	Description	Q.ty
1	19110	STAINLESS STEEL PLATFORM	1

The shipping will be made by ME.BER. free at works and COD.

ME.BER. guarantees the supply of all the spare parts in the shortest time possible, compatibly with warehouse availability and the complexity of the spare part.

Use only ME.BER. spare parts. Replacing broken or worn out parts with low quality parts or personally constructed parts, can irreparably affect the efficiency of the support and will invalidate the product's warranty.

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ME.BER. S.r.l. Unipersonale - Via Langhirano, 270 - 43124 FONTANINI (PR) ITALIA
ME.BER. Srl Unipersonale