ERKA.

THE ORIGINAL

Made in Germany since 1889

USER MANUAL STETHOSCOPES

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XXL Guarantee.
You have trusted in us
FOR 130 YEARS.
Thank you.

Trust must be earned. Through continuity and sustainability. As a fourth generation family company, we at ERKA. stand by our promise of „Quality Made in Germany“. Those who choose an ERKA. product are consciously seeking the highest quality and durability. We are convinced of the quality of our products – as demonstrated by the ERKA. XXL Guarantee. Register your product up to 4 weeks after your purchase at www.myerka.de and get an additional warranty of up to five years – free of charge. NO IFS OR BUTS.

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360° SERVICE
Global expertise
from a single source

Do you have any questions about our products,
their use or maintenance?

For all your needs concerning ERKA. products and
services, our expert ERKA. Service Team is happy
to provide advice and support at the following
phone number **+49 8041 8009-0.**

or by E-mail to service@erka.org.

*Our telephone consultation service can be reached
on weekdays between 8:00 a.m. and 5:00 p.m.

USER MANUAL STETHOSCOPES

English 10 - 15

Français 16 - 21

Deutsch 22 - 27

Italiano 28 - 33

Español 34 - 39

Chinese 40 - 44

Register your device at
www.myerka.de

EXPLANATION OF SYMBOLS | LÉGENDE |
ERKLÄRUNG DER SYMBOLE | EXPLICACIÓN DE SÍMBOLOS |
SPIEGAZIONE DEI SIMBOLI:

Please pay attention to the accompanying User Manual.
Veuillez lire le mode d'emploi ci-joint.
Bitte beiliegende Bedienungsanleitung beachten.
Por favor, tenga en cuenta el manual de instrucciones adjunto.
Si prega di osservare le allegate istruzioni per l'uso.
请注意随附的操作说明书。

This product complies with the Council Directive 93/42/EEC concerning medical devices
Ce produit est en conformité avec la directive 93/42/CEE du Conseil pour les dispositifs médicaux
Dieses Produkt entspricht der Richtlinie 93/42/EWG des Rates über Medizinprodukte
Este producto se corresponde con la directiva 93/42/CEE del Consejo relativa a productos sanitarios.
Questo prodotto è conforme alla Direttiva 93/42/CEE del Consiglio in materia di prodotti medicali.
本产品符合关于医疗器械的第 93/42/ECC 号指令

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USER MANUAL STETHOSCOPES

TABLE OF CONTENTS:

- | | |
|-----------------------------|--------------------------|
| 1. In General | 4. Important Information |
| 2. Product Description | 5. Maintenance and Care |
| 3. Handling the stethoscope | 6. XXL-Warranty |

1. IN GENERAL

Dear Customer,

With this stethoscope, you have acquired a quality product from ERKA. We are delighted that you have expressed your trust in ERKA. quality with your purchase and we thank you for this. We have a large selection of stethoscopes in different models. High quality materials suitable to the task will guarantee you optimal operation. If you have questions regarding the right choice or the operating of your blood pressure monitor, you can direct them to service@erka.org. We would be happy to help you. Our stethoscopes meet the essential requirements of EU Guidelines 93/42/EEC of the Regulation dated 14th June 1993 on medical devices.

Please carefully and thoroughly read this manual before use and follow the instructions for maintenance.

2. PRODUCT DESCRIPTION

PURPOSE

The stethoscopes are primarily used by doctors and trained personnel for auscultation of various organs such as the heart, lungs, intestines and arteries and to assess these sounds. In the measurement of blood pressure, you will use the stethoscope for detection of Korotkoff sounds, to determining the systolic and diastolic blood pressure in adults and children.

SIDE EFFECTS

No side effects are known so far.

FEATURES

The stethoscope is a unit made of chest piece, stethoscope tubing and headband with attached ear tips. ERKA. stethoscopes have model-specific features, which are described in the following.

SENSITIVE

Dual-diaphragm, two-channel tube. Due to the dual-diaphragm, a perfect auscultation of low and high tones is facilitated and thus a wide range of applications in diagnostic medicine for checking heart, lungs and bowel sounds is given.

PRECISE

Dual-diaphragm, funnel, two-channel tube. The dual-diaphragm allows the auscultation of high and low-frequency sounds, while the funnel is used for specially listening to low frequencies (cardiology/heart sounds).

CLASSIC

Dual-diaphragm, single-tube. The auscultation is optimized by the high specific weight of the brass body and will thus reliably avoid background noise.

FINESSE

Convex diaphragm, funnel, two-channel tube. The membrane side is used for auscultation of the entire frequency range, the funnel side for the auscultation of lower frequencies.

FINESSE CHILD

The small variation of the Finesse permits perfect acoustics for the little hearts with small membrane side and funnel side.

FINESSE²

Two membranes, double-channel-tube. The only stethoscope ERKA. of our ERKA-Family with two membrane sides, which make the complete frequency spectrum perfectly audible respectively.

ERKAPHON | CLASSIC SIMPLEX

Convex membrane, single-channel-tube. Particularly suitable for the measurement of blood pressure. The chest piece can be easily slid under the blood pressure cuff.

ERKAPHON CHILD

The smaller version of the Erkaphon. The ergonomically shaped chest piece is perfectly suited to be able to auscultate optimally in inaccessible or very small areas.

ERKAPHON ALU

The aluminium version of the Erkaphon. The stethoscope complies with the basic requirements for a moderate price, whereby the high quality components of Erkaphon series guarantee an excellent auscultation.

ERKAPHON BLACK LINE | CLASSIC LIGHT

The black version of the Erkaphon Aluminum. The chest piece of aluminium is easy and quick to clean

ERKAPHON DUO | FINESSE SIMPLEX

Convex membrane, funnel, single-channel-tube. The membrane side is used for auscultation of the entire frequency range, the funnel side for the auscultation of lower frequencies.

ERKAPHON DUO CHILD | FINESSE SIMPLEX CHILD

The smaller version of the Erkaphon Duo. The small double chest-piece is optimally matched to the children auscultation.

ERKAPHON DUO ALU

The aluminum version of the Erkaphon Duo. The chest piece is quick and easy to clean. The use of efficient materials and durable, compact design make this stethoscope a real economic alternative.

ERKAPHON DUO BLACK LINE | FINESSE LIGHT

The black version of the Erkaphon duo Aluminum satisfies through elegance and design.

3. HANDLING THE STETHOSCOPE

Please note the following points in order to obtain a perfect audio quality:

- Place the stethoscope on the head so that the ear plugs show in the viewing direction. This ensures that the ear plugs are inserted according to the anatomy of the ear canal from behind the ear.
- You can reduce the preload of the stethoscope and thus reduce the pressure on the ear canal by taking the head band between thumb and forefinger and pull the limbs apart slowly.
- If the preload is not sufficient to ensure a tight fit of the stethoscope in your ears, then you can increase the tension by taking the headband between thumb and forefinger and gently squeeze the limbs.
- Ensure regularly that the ear tips are plugged firmly on the ends of the headband.

SWITCHING WITH STETHOSCOPES WITH DOUBLE CHEST PIECE

When using a stethoscope with double chest piece, you must ensure that you have selected the side suitable for the purpose of use by turning the chest piece. Only one side is activated always for the auscultation. The change takes place by turning the chest piece 180° until a click is felt.

4. IMPORTANT INFORMATION

Check the stethoscope before each use as to its proper function.

Do not use this product when it displays visible damage.

Avoid Injury:

- Never use stethoscopes without ear tips.
- Do not use stethoscopes without membrane.

GENERAL INFORMATION

- Interpretation of results should be done only by qualified persons
- Avoid unnecessary vibrations of the instrument
- Avoid damaging the rubber parts with sharp or pointed objects
- Do not put ear plugs too deeply into the ear canal
- Check regularly for contamination and, if necessary carry out the cleaning
- Do not perform heat sterilization
- Do not place in cleaning fluid and protect against contamination, (so that no cleaning liquids or solids enter the cavity system of the stethoscope)
- Only use original ERKA. spare parts
- ERKA. stethoscopes are DEHP and latex-free
- The products should not be placed in a strong magnetic field (except Erkaphon Aluminum) and used, such as: MRI.

STORAGE CONDITIONS/TRANSPORT CONDITIONS

- Storage at room temperature between -10 ° and 40 °C
- Relative humidity less than 85%
- Do not expose the unit to direct UV irradiation, this is particularly applicable for rubber parts

5. INSTRUCTIONS FOR MAINTENANCE

Before each use with direct body contact (patient/examiner), a disinfectant cleaning has to be ensured. Do not immerse or autoclave the stethoscope in liquids. The stethoscope can be cleaned by spray - or disinfectant wipes:

- **Spray disinfection:** Spray all items with the cleaning agent/disinfectant and leave the agent in for some time, according to the manufacturer's instructions. If necessary, wipe with a dry cotton cloth.
- **Wipe disinfection:** Moisten the stethoscope or a cloth with the cleaning agent /disinfectant. Finally wipe the surfaces with slight pressure.

- In order to support these cleaning methods use only clean, soft cotton cloths.

RECOMMENDED CLEANING AGENT

- Fugaten® Spray (lyso-Spray) Alternatively: Esemfix® from Schulke; Foam Cleaner), Promanum® pure from Braun, Meliseptol® Foam pure from Braun.
- For maintaining the elasticity of the tube and other non-metallic components, we recommend a vinyl-care products.

6. XXL WARRANTY

Those who opt for a ERKA. product, is deliberately seeking the highest quality and durability. We are confident in the quality of our products - as demonstrated by the ERKA. XXL Warranty! Register your product after purchase and ensure an additional warranty period of up to 5 years, apart from the statutory warranty of 2 years - at no cost. An overview about the warranty periods for different blood pressure monitors on our homepage.

Each stethoscope is provided with a unique serial number on the chest piece and should be appropriately registered on our website www.myerka.de. At our discretion, we shall remove all defects, which refer to the material or manufacturing defects by repairing or exchanging the device free of charge. Parts that have undergone wear and tear as well as damages due to improper treatment/handling are exempted from the warranty. The warranty expires for interventions that were not performed by authorized service partners, as well as while using replacement parts that are not from ERKA. With proper care, there can be a usage far beyond the warranty period.

If you have any questions, one of our friendly staff will be happy to help you at **+49 08041 8009 0** at any time.

In the event of a warranty claim, please send the complete device with your purchase receipt.

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